3D PRINTING WORKFLOW FOR







## UNIQUE COOPERATION

3D Systems meets Amann Girrbach. 3D printing meets dental workflow. Together, 3D Systems and Amann Girrbach have created a unique system solution for the dental market. By pooling the core competencies of both system partners, 3D printing for the first time enables a dental and fully integrated workflow for the fabrication of dental products and tools.

The combination of the Ceramill workflow with the NextDent 5100 3D printer and the associated 3D Sprint software solution offers the user a plug & play system solution with comprehensive software access. The large selection of compatible NextDent 3D printing materials stands for a maximum range of applications in everyday laboratory work, which is further simplified by clever material management and matching post-processing options.

\_Up to 3x faster printing due to Figure  $4^{\text{TM}}$  technology for high productivity and flexibility

\_Excellent printing results in highest precision due to coordinated workflows, machines, materials and accessories

\_Rapid amortization due to maximum material and indication spectrum as well as low investment and fixed costs





## SIMPLE AND INTUITIVE - FROM DESIGN THROUGH TO PRINTING

The connection of the Ceramill Mind design software to the 3D Sprint nesting software ensures greater efficiency and process stability in everyday laboratory routines. To this end, 3D Systems and Amann Girrbach created a secure interface to enable fast and simple data transfer directly from the design to the nesting software.

At the same time, the steps of the digital workflow were tested from an application engineering angle and adapted to the needs of the dental laboratory. As a result, preset design parameters, automatic orientation of the building object as well as an optimized workflow from design through to printing ensure a smooth and fast production process.

\_End-to-end coordinated Ceramill workflow - for simple, safe and intuitive handling throughout the entire process


\_Safety and time savings in everyday laboratory routine due to the automatic transfer of design data in combination with an intelligent nesting concept

\_Optimally matched CAD parameters for maximum process reliability





Automatic transfer of data from the Ceramill Mind to the 3D Sprint software

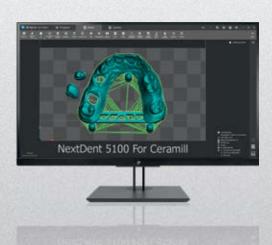



## METICULOUS MATERIAL VALIDATION FOR EXCELLENT RESULTS

During the course of system coordination, all process components such as 3D printer, accessories and post-processing were carefully coordinated with each other. Special attention was paid to the validation of the materials.

The multitude of printable materials and the associated broad spectrum of indications promises maximum flexibility and cost-effectiveness. Predefined and integrated process parameters ensure simple and safe handling as well as assured quality of the restorations. The entire production and post-processing procedure is designed intuitively and reduced to just a few steps in everyday laboratory routine – for maximum efficiency with minimum expenditure of time.

\_Excellent results due to validation of the material and indication spectrum within the overall system

\_Simple and safe handling in the production process due to predefined parameters \_Intelligent material management system with QR code assignment secures the workflow and provides all the information on material status, material quantities and construction job assignment at a glance

### WIDE RANGE OF INDICATIONS AND MATERIALS FOR MAXIMUM ROI

INDICATION	MATERIAL	
Models	NextDent for Ceramill Model 2.0	INTEGRATED
Crowns, bridges, partial dentures	NextDent for Ceramill Cast	INTEGRATED
Crowns, bridges	NextDent for Ceramill C&B MFH	INTEGRATED
Mock-up	NextDent for Ceramill Try-In	VALIDATED
Prosthetic bases	NextDent for Ceramill Denture 3D+	VALIDATED
Drilling templates	NextDent for Ceramill SG	VALIDATED
Individual trays	NextDent for Ceramill Tray	VALIDATED
Bite splints	NextDent for Ceramill Ortho Rigid	VALIDATED
Indirect bonding trays	NextDent for Ceramill Ortho IBT	VALIDATED
Flexible gingiva masks	NextDent for Ceramill Gingiva Mask	VALIDATED





## UNIQUE. FUTURE-PROOF. VALIDATED.

The combination of the familiar Ceramill workflow, the high-speed Figure 4™ technology of the NextDent 5100 for Ceramill, a validated material portfolio and proven post-processing methods provides a simple, coordinated and safe process. This unique workflow enables the user to achieve long-term cost and time savings in resin-based indications. Furthermore, Figure 4™ technology will in future map additional indications and workflows in the NextDent 5100 for Ceramill system.

3D SPRINT NEXTDENT 5100 DATA INPUT CERAMILL MIND PRINTABLE MATERIALS CLEANING LIGHT CURING PROCESS VALIDATED RESULTS for Ceramill for Ceramill

Digitization of the initial situation by means of intraoral or model scans.

Defined material selection and design parameters for maximum process reliability.

flow, auto support and orientation function for optimal printing results. reproducible workflows.

Guided fabrication work- Sophisticated and coordinated barcode system ensures efficient and

Comprehensive material Defined ultrasonic portfolio for maximum flexibility and economical use of the 3D printer.

cleaning times and processing recommendations for optimum material quality.

Light curing device adapted to the material with defined curing times for precise, reproducible and standard-compliant results.



## THE HIGH-SPEED 3D PRINTER FOR DENTAL MATERIALS

The high-speed 3D printer NextDent 5100 for Ceramill with the groundbreaking Figure  $4^{\text{TM}}$  technology stands for high productivity at first-class speed at a price that is affordable for practically all dental laboratories.

In addition to perfect restoration results, the 3D printer scores with enormous time savings due to the revolutionary Figure  $4^{TM}$  technology and the oxygen-permeable membrane in the resin tray. The low pull-off forces between the membrane and the construction platform ensure a distortion-free printing process and excellent results. A gentle separation process enables the use of filigree support structures, which can not only be separated easily and without tools in the post-processing process, but also in a time-saving manner.

And last but not least, the NextDent 5100 for Ceramill convinces with its simple single-hand handling. The integrated color display can also be operated with gloves and both the construction platform and the resin tray are easily accessible and user-friendly during application.

\_Up to 3x faster printing due to Figure  $4^{\text{TM}}$  technology for high productivity and flexibility

\_Low pull-off forces due to an oxygen-permeable membrane ensure a long service life of the resin tray, easy separation of the support structures and distortion-free results

\_Easy handling due to touch screen and single-hand operation of accessories such as the construction platform and resin tray

#### **TECHNICAL DATA**

Weight	34.5 kg / 54.4 kg (with pedestal)
Dimensions (DxWxH)	42.6 x 48.9 x 97.1 cm / 68.1 x 70.4 x 135.6 cm (with pedestal)
Printing time	Up to 12.1 cm/h depending on layer thickness
Layer thickness	30-100 μm
Resolution	1920 x 1080 pixels
Construction platform (DxWxH)	12.4 x 7.0 x 19.6 cm





## WORKFLOW ACCESSORIES - FOR TRUSTED AND REPRODUCABLE RESULTS

### LC-3D MIXER

The LC-3D Mixer is a roller/tilting stirring device for stirring of 3D printing materials before use. Due to a defined mixing process with fixed stirring times it ensures stable material conditions before printing. Overall the best solution for constant and homogenous printing results.

VALUE
AC 100-240 V, 50/60 Hz
10 W
250 V, T2A
41 x 27 x 10 cm
4 kg



### LC-3D PRINT BOX

The LC-3D Print Box is a revolutionary UV light box, suitable for post-curing 3D printing materials. The LC-3D Print Box is equipped with 12 UV light bulbs strategically placed inside the box. This ensures that a product is illuminated from all sides, which results in a quick and uniform curing cycle. The spacious interior resin allows you to easily cure multiple products at once. In addition, the box has enough space to place an articulator inside. The LC-3D Print Box is integrated in the Ceramill process and ensures validated and constant results in the Ceramill process chain.

VALUE
AC 110-230V, 50/60 Hz, 2.6A/1.3A
10 W
T2.0A, AC250V
41 x 44 x 38 cm
22 kg





## THE MULTITUDE OF MATERIALS OF THE NEXTDENT 5100 FOR CERAMILL

The large selection of printable materials and the associated broad spectrum of dental indications ensure maximum flexibility and economical use of the NextDent 5100 for Ceramill in everyday laboratory work.



#### INTEGRATED

NextDent for Ceramill Model 2.0
3D print material for models

\_Simple and fast fabrication of dental models \_Perfect fit of the restorations due to highest precision

\_High level of detail and surface quality

PROPERTY	REQUIREMENT	RESULT	ISO STANDARD
Flexuralmodulus	≥ 1500MPa	1980	ISA 178
HardnessshoreD	≥80 ShoreD	84	ISO 178

Colors: peach, white, gray, ochre



#### INTEGRATED

NextDent for Ceramill Cast 3D print material for casting / pressing technique

\_High stability of the printed structures \_Burns out residue-free for optimum results

PROPERTY	REQUIREMENT	RESULT	ISO STANDARD
Flexuralmodulus	≥1500MPa	1980	ISA 178
HardnessshoreD	≥80 ShoreD	84	ISO 178

Colors: Violet



#### INTEGRATED

NextDent for Ceramill C&B MFH 3D print material for temporary crowns and bridges

\_High strength and wear resistance \_Natural esthetics due to different colors and coordinated translucency

PROPERTY	REQUIREMENT	RESULT	ISO STANDARD
Flexural strength	≥ 50MPa	107	ISA 10477
Sorption	≤ 60 µg/mm³	54	ISO 10477
Solubility	≤ 12.5 µg/mm³	5.9	ISO 10477
Biocompatibility	according to ISO Standard	comply	ISO 109993-1

Colors: BL, N1, N1.5, N2, N2.5, N3 and T1



#### VALIDATED

NextDent for Ceramill Denture 3D+ 3D print material for denture bases

\_Significantly lower shrinkage than standard PMMA materials for best fit results \_Available in different colors for individual results

PROPERTY	REQUIREMENT	RESULT	ISO STANDARD
Ultimate flexural strength	≥ 65MPa	84	ISA 20795-1
Sorption	≤ 32 µg/mm³	28	ISO 20795-1
Solubility	≤ 1.6 µg/mm³	0.1	ISO 20795-1
Residual monomer	≤ 2.2 % (w/w)	< 0.1	ISO 20795-1
Biocompatibility	according to ISO Standard	comply	ISO 20795-1

Colors: Dark pink, light pink, pink opaque, red pink and translucent pink.



#### VALIDATED

NextDent for Ceramill Try-In 3D print material for try-in prosthetics

\_Best choice for checking digitally designed prosthetic bases with individually designed tooth setups Biocompatible MD Class I material

PROPERTY	REQUIREMENT	RESULT	ISO STANDARD
Flexuralmodulus	≥ 1500MPa	2043	ISO 20795-1
Biocompatibility	according to ISO Standard	comply	ISO 20795-1

Colors: TIO, TI1 and TI2





#### VALIDATED

#### NextDent for Ceramill SG (Surgical Guide) 3D print material for drilling templates

\_Easy insertion of the drilling sleeves due to highest precision \_Can be sterilized with standardized autoclaving protocols \_Biocompatible MD Class I material

PROPERTY	REQUIREMENT	RESULT	ISO STANDARD
Ultimate flexural strength	≥ 50 MPa	85	ISA 20795-1
Flexuralmodulus	≥ 1500 MPa	2118	ISO 20795-1
Residualmonomer	≤ 2.2 % (w/w)	< 0.1	ISO 20795-1
Biocompatibility	according to ISO Standard	comply	ISO 10993-1

Color: orange translucent



#### VALIDATED

#### NextDent for Ceramill Ortho Rigid 3D print material for splints

\_Fast fabrication of precisely fitting splints \_Biocompatible MD Class I material

PROPERTY	REQUIREMENT	RESULT	ISO STANDARD
Ultimate flexural strength	≥ 50 MPa	78	ISA 20795-2
Sorption	≤ 32 µg/mm³	0.8	ISO 20795-2
Solubility	≤ 5.0 µg/mm³	0.8	ISO 20795-2
Residualmonomer	≤ 5.0 % (w/w)	< 0.1	ISO 20795-2
Biocompatibility	according to ISO Standard	comply	ISO 10993-1

Color: blue transparent



#### VALIDATED

## NextDent for Ceramill Tray 3D print material for individual trays

\_Quality impressions with high precision in next to no time \_Compatible with all types of impression materials \_Biocompatible MD Class I material

PROPERTY	REQUIREMENT	RESULT	ISO STANDARD
Ultimate flexural strength	≥ 50MPa	81	ISA 20795-1
Flexuralmodulus	≥ 1500MPa	2015	ISO 20795-1
Residualmonomer	≤ 2.2 % (w/w)	< 0.1	ISO 20795-1
Biocompatibility	according to ISO Standard	comply	ISO 10993-1

Colors: blue, pink



#### VALIDATED

#### NextDent for Ceramill Gingiva Mask 3D print material for gingival masks

\_Easy fabrication of flexible parts such as gingival masks \_Best results in combination with Model 2.0

PROPERTY	REQUIREMENT	RESULT	ISO STANDARD
ShoreA hardness	60-75	1980	ISO 10139-2
Elongation at break	40-60%	53	ISO 527-1 ISO 527-2

Color: pink



#### VALIDATED

#### NextDent for Ceramill Ortho IBT 3D print material for orthodontic transfer splints

\_Easy positioning and application of orthodontic brackets due to precise and flexible splint material \_Biocompatible MD Class I material

PROPERTY	REQUIREMENT	RESULT	ISO STANDARD
ShoreA hardness	75-90	85	ISO 101392
Elongation at break	10-20 %	17	ISO 527-1 ISO 527-2
Biocompatibility	according to ISO Standard	comply	ISO 10993-1

Color: transparent



# ORDERING INFORMATION

NextDen	t 5100 for Ceramill	
181600	NextDent 5100 - AG	
181602	Resin Tray - ND 5100	1 pc.
181604	Printer Platform - ND 5100	2 pcs.
181605	Filter, Carbon - ND 5100	1 pc.
181606	Filter, Intake - ND 5100	1 pc.
181607	Punch Tool - ND 5100	1 pc.
181608	Platform Scraper - ND 5100	1 pc.
181609	Scouring Brush - crimp brass - ND 5100	1 pc.
181610	Part Cleaning Brush - ND 5100	1 pc.
181611	Resin Mixer - ND 5100	1 pc.
Printer F Stand for	Pedestal NextDent 5100 for Ceramill	
181601	Printer Pedestal	1 pc.
	t LC-3D Print Box ng furnace for 3D printing materials NextDent LC-3D Print Box - AG	1 pc.
	t LC-3D Mixer levice for mixing 3D printing materials	
181810	NextDent LC-3D Mixer - AG	1 pc.
Resin Tra	<b>ay Kit</b> y including storage garage kit	
181602	Resin Tray - ND 5100	1 pc.

Storage	Carago Kit	
	Garage Kit storage of the filled resin tray	
181603	Storage Garage Kit - ND 5100	1 pc.
Printer F Building		
181604	Printer Platform - ND 5100	1 pc.
Filter, Ca	arbon	
181605	Filter, Carbon - ND 5100	1 pc.
Filter, In	take	
181606	Filter, Intake - ND 5100	1 pc.
D T		
Punch To Separation	ooi ng aid for printed objects	
	Punch Tool - ND 5100	1 pc.
101001	Tallet 1001 TIP STOC	
<b>Platform</b> Cleaning	Scraper tool	
	Platform Scraper - ND 5100	1 pc.
<b>Scouring</b> Cleaning	g Brush - Crimp Brass tool	
181609	Scouring Brush - crimp brass - ND 5100	1 pc.
Part Cle Cleaning	aning Brush tool	
181610	Part Cleaning Brush - ND 5100	1 pc.



# ORDERING INFORMATION

Resin Mixer Material mixer for res	in tray		NextDent Ortho IBT 3D print material for	orthodontic transfer splints	
181611 Resin Mixe	r - ND 5100	1 pc.	NPAGOICL01000	NextDent Ortho IBT	1000 g
NextDent Cast 3D print material for o	casting / pressing technique NextDent Cast / Purple	1000 g	NextDent Ortho Rig 3D print material for NPAGORBL01000		1000 g
NextDent C&B MFH 3D print material for t	temporary crowns and bridges		NextDent Ortho SG 3D print material for	drilling templates	
NPAGCMT101000	NextDent C&B MFH / T1	1000 g	NPAGSGOR01000	NextDent SG / Orange	1000 g
NPAGCMN101000	NextDent C&B MFH / N1	1000 g			
NPAGCMN1.501000	NextDent C&B MFH / N1.5	1000 g	NextDent Tray		
NPAGCMN201000	NextDent C&B MFH / N2	1000 g	3D print material for		
NPAGCMN2.501000	NextDent C&B MFH / N2.5	1000 g	NPAGTRBL01000	NextDent Tray / Blue	1000 g
NPAGCMN301000	NextDent C&B MFH / N3	1000 g	NPAGTRPI01000	NextDent Tray / Pink	1000 g
NPAGCMBL01000	NextDent C&B MFH / BL	1000 g	N		
			NextDent Try-In 3D print material for	trv-ins	
NextDent Gingiva Ma			NPAGTITIO01000	NextDent Try-In / TIO	1000 g
3D print material for o		1000	NPAGTITIO1000	NextDent Try-In / TII	1000 g
NPAGGMPI01000	NextDent Gingiva Mask	1000 g	NPAGTITI201000	NextDent Try-In / TI2	1000 g
NextDent Model 2.0			W AOTHIZO1000	NextBellt Try III, Tiz	10009
3D print material for r			NextDent Denture 3	D+	
NPAGM2PE01000	NextDent Model 2.0 / Peach	1000 g	3D print material for prosthetic bases		
NPAGM2WH01000	NextDent Model 2.0 / White	1000 g	NPAGD3DP01000	NextDent Denture 3D+ / Dark Pink	1000 g
NPAGM2GR01000	NextDent Model 2.0 / Grey	1000 g	NPAGD3LP01000	NextDent Denture 3D+ / Light Pink	1000 g
NPAGM20K01000	NextDent Model 2.0 / Ochre	1000 g	NPAGD30P01000	NextDent Denture 3D+ / Opaque Pink	1000 g
			NPAGD3TP01000	NextDent Denture 3D+ / Translucent Pink	1000 g
			NPAGD3RD01000	NextDent Denture 3D+ / Red Pink	1000 g

#### **AUSTRIA (HEADQUARTERS)**

Amann Girrbach AG Koblach, Austria Fon +43 5523 62333-105 austria@amanngirrbach.com

#### GERMANY

Amann Girrbach GmbH Pforzheim, Germany Fon +49 7231 957-100 germany@amanngirrbach.com

#### NORTH AMERICA

Amann Girrbach North America, LP Charlotte, NC, U.S.A. Fon +1 704 837 1404 america@amanngirrbach.com

#### BRASIL

Amann Girrbach Brasil LTDA Curitiba, Brasil Fon +55 41 3287 0897 brasil@amanngirrbach.com

#### ASIA

Amann Girrbach Asia PTE LTD. Singapore, Asia Fon +65 6592 5190 singapore@amanngirrbach.com

#### CHINA

Amann Girrbach China Co., Ltd. Beijing, China Fon +86 10 8886 6064 china@amanngirrbach.com